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Geachte klant,

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DIRECTIONS FOR USE:

Dear customer!

We congratulate you with the purchase of our unique, brand-protected product **Vagitherm**[®]. We request that you spend a few minutes carefully reading through the following instructions. With the correct use the **Vagitherm**[®] can positively influence your well-being and help you to relieve discomfort around your menstruation period. Pay special attention to the safety instructions.

WHAT EXACTLY IS THE **Vagitherm**[®] ?

Vagitherm[®] is aimed at women and young girls in their fertile age, who struggle sporadically or regularly, before or also during their menstruation, with pain in the lower body, caused by menstruation. The **Vagitherm**[®] is a HEAT-ACCUMULATOR made from a special artificial rubber. The **Vagitherm**[®] can be considered as a useful supplement to conventional treatment methods and is very easy to maintain.

HOW DOES THE **Vagitherm**[®] WORK?

The **Vagitherm**[®] works just like the well-known externally placed "hot water bottle". However, with the important and innovative difference that the cramp-soothing, relaxation-enhancing heat is brought directly in front of the actual problem area, namely the uterus (womb). Here the gentle heat can work as a relaxation-enhancer and analgesic.

HOW IS THE **Vagitherm**[®] USED IN THE CORRECT WAY?

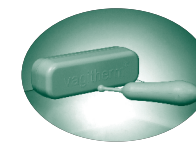
In principle the **Vagitherm**[®] is heated in a water bath. For this, take a sufficiently large pan and fill this with tap water. Now put the HEAT-ACCUMULATOR into the water; this must be immersed in water to a depth of at least three fingers, and this needs to be boiled for approximately twenty minutes. Then dispose of the water and remove the HEAT-ACCUMULATOR from the pan by taking hold of the spherical pulling loop. Then, with the inside of your hand, lightly check the actual temperature of the HEAT-ACCUMULATOR. If this seems too high for you to insert – which will normally be the case – then allow the HEAT-ACCUMULATOR to cool down in the open air for a few minutes before gradually checking the temperature again - this time against your cheek or on the inside of your upper leg or the inside of your wrist. These are locations where the human body reacts particularly sensitively to temperature changes. As a rule the HEAT-ACCUMULATOR is cooled down, at room temperature, to about 45° Celsius after approximately ten minutes.

As soon as you feel that the HEAT-ACCUMULATOR is still "pleasantly warm", but no longer "too hot", insert it into your vagina, just like an ordinary tampon. Please pay attention that after insertion only the pulling string with the broad spherical loop protrudes from the vagina. Only in this way has the HEAT-ACCUMULATOR been inserted correctly. You will immediately notice how the soothing heat flows through your lower body. It is advisable to support this positive effect by granting yourself a few minutes of relaxation. Perhaps you can lay down on the sofa or on the bed, with your eyes closed, or finish reading a book, or indeed whatever you had in mind to do for a while. . . .

As soon as you have the feeling that the heat distribution is clearly reducing – this is also different in each person – you must carefully remove the HEAT-ACCUMULATOR by slowly pulling the string, just as you would remove an ordinary tampon.

HOW IS THE **Vagitherm**[®] CLEANED?

After use you must thoroughly clean the HEAT-ACCUMULATOR with soap and water or with a washing up liquid, and then dry it. You must under no circumstances use aggressive detergents or solvents which contain benzene or white spirit for this. For storage use the supplied plastic box with the ori-

ginal **Vagitherm**[®] lettering. This way your personal **Vagitherm**[®] remains well protected against dust and environmental influences.

WHAT MUST BE PAYED ATTENTION TO AT ALL TIMES?

The **Vagitherm**[®] is not suitable for young girls and women of whom the hymen ("virginal membrane") is still intact, for example because they have not had sexual intercourse. In this case the use of the **Vagitherm**[®] on the basis of its shape and size, could lead to defloration (deflowering), which may not in any way be desired by the user. The manufacturer and distributor are not responsible for the consequences of the disregard of this explicit warning!

Before the first use of the **Vagitherm**[®] you must boil the HEAT-ACCUMULATOR for approximately 20 minutes in the water bath. Personal heat sensitivity varies a lot, as hundreds of users have already proven. You must not vary the twenty minute boiling time phase because of hygiene reasons. However, you can experiment a little with the cooling down phase. You will quickly discover "your personal - **Vagitherm**[®] temperature" and standardise this for yourself. Even after boiling for a longer period the initial temperature of the HEAT-ACCUMULATOR remains below approximately 60° Celsius, so that you only need to remember the last cooling down time. This so that, in the future, you can always find that temperature that is most pleasant for you.

HOW TO MAINTAIN THE **Vagitherm**[®] ?

When you closely observe the instructions for use and cleaning, then the **Vagitherm**[®] needs no special care. Thoroughly clean the HEAT-ACCUMULATOR after each use as described and do not deviate from the instructions for use. You can also clean the supplied box with soap and water, should this be necessary.

IMPORTANT INSTRUCTION!

Only use the **Vagitherm**[®] when it has already been determined by your physician that your complaints are the familiar consequences of your own menstruation and when your physician does not fundamentally oppose the use of heat, for example, the use of an external hot water bottle. In the event of unclear complaints you must in any case first contact your general practitioner and/or gynaecologist!

SIDE-EFFECTS:

There are no side-effects known at the moment. However, in some people the most diverse substances which come into contact with the body can cause allergic reactions. Should you detect, in combination with the use of **Vagitherm**[®], physical reactions in yourself, such as reddened skin or a rash, then you must discuss the continued use of the **Vagitherm**[®] with your physician.

We hope that the **Vagitherm**[®] will help you relieve your complaints month on month in a special, natural way, by means of soothing heat!

Should you have any suggestions or questions regarding the **Vagitherm**[®], then we are happy to be at your disposal. For further information you can also visit our internet pages on the internet via the websites: www.vagitherm.nl, www.vagitherm.be, www.vagitherm.de and www.vagitherm.com

Information situation: October 2008
Distribution: Trophax B.V. P.O. box 134
3330 AC Zwijndrecht – the Netherlands
Item code 900600
UAC code 8717624040139
Z-Index (NL) 15413225
PZN (DL) 4686023

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Distribution : Trophax B.V. Postbus 134